

About GaussFlow™

GaussFlow is a lightweight, comfortable device worn on the ear(s) like headphones that may help restore sudden sensorineural hearing loss (SSHL) using groundbreaking magnetic technology.


Gauss magnetic therapy is a groundbreaking approach that uses static magnetic fields to increase micro-circulatory blood flow, bringing oxygenated blood and medication to an injured site, reducing inflammation, and promoting healing. This process helps to support recovery from sudden sensorineural hearing loss.

The GaussFlow headband fits most adults. For best results, wear GaussFlow as often as possible—anytime, day or night. You can even wear it while you sleep. Read complete user guide before use.

IMPORTANT

Before using GaussFlow, read disclaimer information on the back panel.

What's Included?

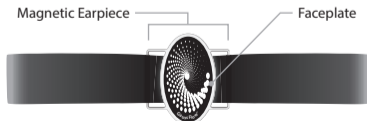

GaussFlow Headband



User Guide

Disclaimer

GaussFlow™ is not to be used as a substitute for medical advice, diagnosis, or treatment of any health condition or problem. Users of this product should not rely on information provided in these instructions for their own health problems. Any questions regarding your own health should be addressed to your own physician or other healthcare provider.

Do not use Magnetic Therapy if:

1. You are pregnant or thinking of becoming pregnant.
2. You have metal or implanted medical devices in your body, including:
 - Aneurysm clips or coils
 - Stents
 - Implanted stimulators
 - Implanted vagus nerve or deep brain stimulators
 - Implanted electrical devices, such as pacemakers or medicine pumps
 - Electrodes for monitoring brain activity
 - Cochlear implants for hearing
 - Magnetic implants
 - Bullet fragments
 - Other metal devices or objects implanted in your body



www.gaussflow.com

A proud member of

GAUSS
PRODUCTS



EAR TECHNOLOGY CORPORATION®

1-800-327-8547 | www.eartech.com

GAUSSFLOW USR GDE 0824 Rev A

USER GUIDE



for SUDDEN SENSORINEURAL HEARING LOSS

EAR TECHNOLOGY CORPORATION®


PATENTS PENDING

GaussFlow™ User Guide


GaussFlow can be used on one or both ears. For a single ear, use one magnetic earpiece with the headband. For both ears, use one earpiece on each ear with one headband.

Continue medication as prescribed while wearing GaussFlow.


1. Place the oval disc over affected ear(s).

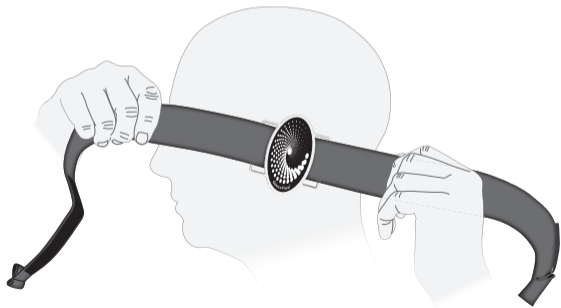


2. Adjust the headband strap until it is comfortable to wear.

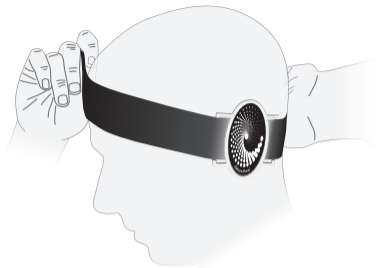

3. Secure the headband with the hook and loop fastener.

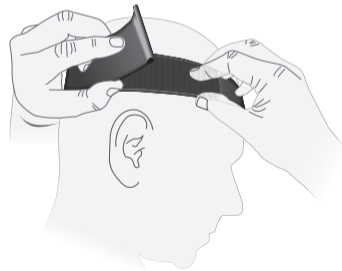

4. We recommend wearing GaussFlow a minimum of two hours per day, though wearing it for longer periods may be more helpful.

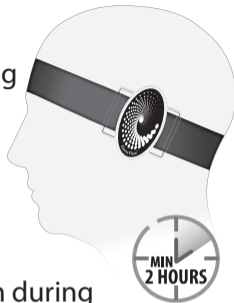

GaussFlow can be worn during the day or night—even while you sleep.

Use GaussFlow at least four weeks before deciding if the hearing loss is improved.



Wash the headband as needed. Remove magnetic earpiece(s) prior to washing.